**THUASNE**

## Ortel C3




<b>fr</b>	Collier cervical rigide.....	6
<b>en</b>	Rigid cervical collar.....	7
<b>de</b>	Feste Halskrause.....	8
<b>nl</b>	Stijve nekbrace.....	9
<b>it</b>	Collare cervicale rigido.....	11
<b>es</b>	Collarín cervical rígido.....	12
<b>pt</b>	Colar cervical rígido.....	13
<b>da</b>	Stiv nakkekrave.....	14
<b>fi</b>	Jäykkä kaulatuki.....	15
<b>sv</b>	Rigid nackkrage.....	17
<b>el</b>	Αυθενικό κολάρο σκληρό.....	18
<b>cs</b>	Pevný krční límec.....	19
<b>pl</b>	Kołnierz sztywny do unieruchomienia szyi.....	20
<b>lv</b>	Stingra kakla ortoze.....	22
<b>lt</b>	Standus kaklo įtvaras.....	23
<b>et</b>	Jäik emakakaela krae.....	24
<b>sl</b>	Opornica za vrat, toga.....	25
<b>sk</b>	Anatomicky tvarovaná penová opora.....	27
<b>hu</b>	Kemény nyakmerevítő.....	28
<b>bg</b>	Твърда цервикална яка.....	29
<b>ro</b>	Guler cervical rigid.....	31
<b>ru</b>	Жесткий шейный воротник.....	32
<b>hr</b>	Tvrda ortoza za vratnu kralježnicu.....	33
<b>zh</b>	硬质颈圈.....	35
<b>ar</b>	طوق للعنق صلب.....	37

	1	28 - 33 cm
	2	34 - 39 cm
	3	40 - 46 cm

<b>fr</b>	Tour de cou	<b>lv</b>	Kakla apkārtmērs
<b>en</b>	Neck circumference	<b>lt</b>	Kaklo apimtis
<b>de</b>	Halsumfang	<b>et</b>	Reieübermõõt
<b>nl</b>	Omtrek van de nek	<b>sl</b>	Obseg vratu
<b>it</b>	Circonferenza del collo	<b>sk</b>	Obvod krku
<b>es</b>	Circunferencia del cuello	<b>hu</b>	Nyak kerülete
<b>pt</b>	Perímetro do pescoço	<b>bg</b>	Размер на яката
<b>da</b>	Halsens omkreds	<b>ro</b>	Circumferința gâtului
<b>fi</b>	Kaulanpääryn	<b>ru</b>	Обхват шеи
<b>sv</b>	Runt halsen	<b>hr</b>	Opseg vrata
<b>el</b>	Περίμετρος λαιμού	<b>zh</b>	颈围
<b>cs</b>	Obvod krku	<b>ar</b>	محيط العنق
<b>pl</b>	Obwód szyi		

<b>fr</b>	Stabilisation	Immobilisation
<b>en</b>	Stabilisation	Immobilisation
<b>de</b>	Stabilisierung	Ruhigstellung
<b>nl</b>	Stabilisatie	Immobilisatie
<b>it</b>	Stabilizzazione	Immobilizzazione
<b>es</b>	Estabilización	Inmovilización
<b>pt</b>	Estabilização	Imobilização
<b>da</b>	Stabilisering	Støtte
<b>fi</b>	Stabiloi	Immobilisoi
<b>sv</b>	Stabilisering	Immobilisering
<b>el</b>	Σταθεροποίηση	Ακινητοποίηση
<b>cs</b>	Stabilizace	Znehybnění
<b>pl</b>	Stabilizacja	Unieruchomienie
<b>lv</b>	Stabilizēšana	Imobilizācija
<b>lt</b>	Stabilizavimas	Imobilizavimas
<b>et</b>	Stabiliseerimine	Liikumatuks muutmine
<b>sl</b>	Stabiliziranje	Imobilizacija
<b>sk</b>	Stabilizácia	Imobilizácia
<b>hu</b>	Stabilizálás	Rögzítés
<b>bg</b>	Стабилизация	Обезгвужване
<b>ro</b>	Stabilizare	Imobilizare
<b>ru</b>	Стабилизация	Иммобилизация
<b>hr</b>	Stabilizacija	Imobilizacija
<b>zh</b>	稳定	固定
<b>ar</b>	الثبت	التثبيت

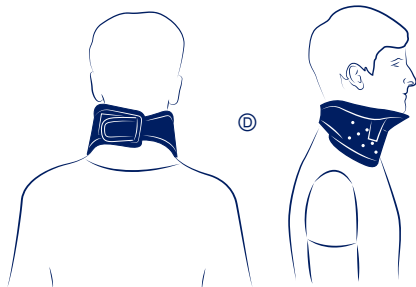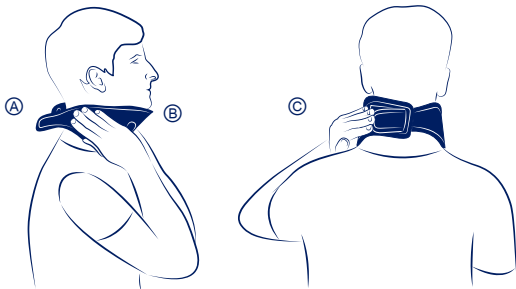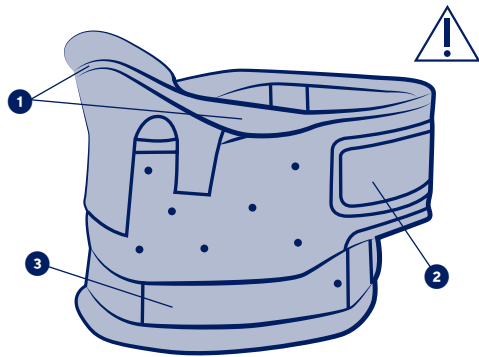**MD**

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### Description/Destination

Anatomically-shaped chin support. ①

Precise self-fastening height adjustment (9 cm to 13 cm). ②

Precise self-fastening adjustment. ③

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

### Composition

Cotton - polyamide - polyester - polyethylene - polyurethane.

### Properties/Mode of action

Stabilization of the cervical spine.

Immobilization of the cervical spine in severe injuries.

### Indications

Cervical radiculopathy.

Cervical spondylosis.

Stable cervical fractures.

Post-operative use.

Whiplash.

### Contraindications

Do not apply the product in direct contact with broken skin.

Do not use in the event of known allergy to any of the components.

Do not use in the event of traumatic injuries: severe sprain, unstable fracture.

Do not use for prolonged wear without medical supervision.

Do not use the product if the diagnosis has not been confirmed.

### Precautions

Strictly comply with your healthcare professional's prescription and recommendations for use.

In the event of discomfort, significant hindrance, pain, abnormal sensations, remove the device and consult a healthcare professional.

Choose the appropriate size to fit the patient, referring to the size chart.

Verify the product's integrity before every use.

It is recommended that a healthcare professional supervises the first application.

For hygiene and performance reasons, do not re-use the product for another patient.

Do not use the device if it is damaged.

Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

It is recommended to adequately tighten the device to achieve support without excessive compression.

Do not wear the product in a medical imaging machine.

Do not wear the product while sleeping.

Do not wear the product when driving a vehicle.

### Undesirable side-effects

This device can cause skin reactions (redness, itching, burns, blisters...) or wounds of various degrees of severity.

Wearing a cervical collar may cause some weakness of the neck muscles.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

### Instructions for use/Application

Open the self-fastener.

Place the collar around the neck. Ⓐ

The chin should rest on the chin support. Ⓑ

Close the self-fastener at the nape of the neck. Ⓒ

Ensure the strap is not fastened too tight. Ⓓ

Adjust the height if necessary.

### Care

See product label. Close the self-fastening tabs before washing. Hand wash.

Do not dry clean. Do not use detergents, fabric softeners or aggressive products (products containing chlorine). Do not tumble-dry. Do not iron. Dry away from any direct heat source (radiator, sun, etc.). Dry flat.

### Storage

Store at room temperature, preferably in the original packaging.

### Disposal

Dispose of in accordance with local regulations.

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©Thuasne - 2029101 (2021-03)
Thuasne SAS - SIREN/RCS Nanterre 542 091 186 - capital 1 950 000 euros