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COMFORT CRUTCH

Description/Destination

Comfort crutch (walking aid)
This device helps temporarily or permanently mobility impaired individuals to maintain their freedom of movement by providing them with an additional point of support. It can be used inside and outside.

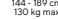
Main components

Tubes : aluminum.
Forearm support cuff: polypropylene.
Handgrip: polypropylene + thermoplastic.
Ferrule: rubber.

Indications

- Post-operative and post-traumatic support.
- Muscle weakness, neuromuscular disorders, stability problems, impaired balance, arthritis of the lower limbs.
- Degenerative conditions of the lower limbs.

Contraindications

 Not suitable for patients weighing > 130 kg.
 Not suitable for patients measuring < 144 cm or > 189 cm.

Do not use in the event of known allergy to any of the components.

Precautions

 **Important warnings**

Verify the product’s integrity before every use. Do not use the device if it is damaged. It is recommended that a health care professional supervises the first use. Follow the advice of the health care professional who prescribed or supplied the product and consult them in case of doubt. Before each use of this product, check that it is stable and rigid. In case of any visible deterioration or significant wear of the ferrule, contact a specialized dealer or health care professional to have it replaced. Do not make any mechanical changes to the device, this could compromise its safety. Only original spare parts and accessories have to be used. Take care when using the device on slippery surfaces (wet, snow-covered or icy floors or wet tiles), uneven or unstable surfaces (gravel, sand, leaves). Do not leave exposed to direct sunlight for a long period of time.

Undesirable side-effect

- In case of incorrect use or overuse of the device: muscle fatigue, inflammation or injury. In case of incorrect height adjustment: Skin reactions (redness, rashes, irritation, blisters) or wounds of varying severity, shoulder joint strain, pain in the hand, wrist, forearm or arm (biceps), hematomas and bruises, etc.
 - Danger of falling or loss of balance, which can lead to serious injuries.
- Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use

The height adjustment of a walking aid/crutch must be done with reference to the height of the patient. Incorrect height adjustment can impair performance and place too much strain on the user.

Walking aid/crutch height adjustment :

After adjustment, when standing up straight with one hand on the handle, the arm should be bent at an angle of 30° (Fig. A). Adjustment by push-buttons: with a finger on each push-button, press both buttons simultaneously while pulling the interior tube in order to adjust the height. Make sure that the walking aid/crutch is locked in position by gradually pressing it with increasing force on the ground.

Forearm support height adjustment (if available) :

Proceed in the same way as for the lower part.

Make sure that both walking aids/crutches have been adjusted to the same height.

Use :

The walking aid/crutch must be placed as vertical as possible (Fig. A) on the ground. Never place the walking aid/crutch too far to the side or too far in front as this creates a risk of falling. When in use, grasp the device’s handle completely and firmly. In case of a fall: let the walking aid/crutch aid fall to the side so that you do not fall on it.

 **Important warnings**

Always keep one hand on the handrail when going up or down stairs (Fig. B and C). If possible, ask somebody for assistance. When it is necessary to take stairs without a handrail, only do so with extreme caution. Do not hang purses, carrier bags or other objects on your walking aid (Fig. D). Only use the device as intended and not for any other use: pulling, lifting or moving objects (Fig. E).

Maintenance and care

Maintenance
If any part of the device becomes detached or loose, contact a specialized dealer or health care professional. If necessary, replacement parts and original accessories are available from specialized dealers for the whole range of Thuasne products.

Care

Clean with a mild cleaning agent and a soft cloth. Do not clean the handle or the ferrule with greasy substances, this could create a slip hazard. If necessary, use a mild contact disinfectant or 70° alcohol.

Storage

Store at room temperature.

Disposal

Dispose of in accordance with local regulations.

Warranty

In the European Union, the product is covered by the two-year European legal guarantee of conformity.

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