**THUASNE**

# MalleoPro Activ

<b>fr</b>	Chevillère élastique proprioceptive renforcée.....	5
<b>en</b>	Reinforced proprioceptive elastic ankle brace.....	6
<b>de</b>	Verstärkte elastische propriozeptive Knöchelbandage.....	7
<b>nl</b>	Verstevigde proprioceptieve elastische enkelbrace.....	8
<b>it</b>	Cavigliera elastica proprioceettiva rinforzata.....	10
<b>es</b>	Tobillera elástica proprioceptiva reforzada.....	11
<b>pt</b>	Tornozoleira elástica proprioceptiva reforçada.....	12
<b>da</b>	Forstærket proprioceptisk, elastisk ankelbind.....	13
<b>fi</b>	Vahvistettu proprioseptiivinen joustava nilkkatuki.....	14
<b>sv</b>	Förstärkt proprioceptiv elastisk ankelortos.....	16
<b>el</b>	Ενισχυμένη ελαστική ιδιοδεκτική επιστραγαλίδα.....	17
<b>cs</b>	Elastická proprioceptivní zesílená kotníková ortéza.....	18
<b>pl</b>	Wzmocniona elastyczna proprioceptywna orteza skokowo-stopowa.....	19
<b>lv</b>	Proprioceptīvs elastīgs potītes pārsējs ar pastiprinājumu.....	20
<b>lt</b>	Tamprus sutvirtintas propriocepriją gerinantis čiurnos įtvaras.....	22
<b>et</b>	Proprioseptiivne elastne tugevdatud hüppeliigesetugi.....	23
<b>sl</b>	Ojačena elastična proprioceptivna opornica.....	24
<b>sk</b>	Vystužená elastická proprioceptívna členková ortéza.....	25
<b>hu</b>	Megerősített, proprioceptív rugalmas bokarögzítő.....	26
<b>bg</b>	Подсилена проприоцептивна еластична шина за глезен.....	28
<b>ro</b>	Glezniere elastică proprioceptivă ranforsată.....	29
<b>ru</b>	Усиленный эластичный проприоцептивный ортез.....	30
<b>hr</b>	Elastični proprioceptivni steznik za gležanj s ojačanjem.....	31
<b>zh</b>	加强的弹力本体感护踝.....	33
<b>ar</b>	دعامة كاحل مرنة متعلقة باستقبال الحس العميق مقواة.....	35

	1	28 - 29 cm
	2	30 - 31 cm
	3	32 - 33 cm
	4	34 - 35 cm
	5	36 - 38 cm

<b>fr</b>	Tour du cou-de-pied
<b>en</b>	Instep circumference
<b>de</b>	Ristumfang
<b>nl</b>	Omvang van de wreef
<b>it</b>	Giro del collo del piede
<b>es</b>	Circunferencia del empeine
<b>pt</b>	Perímetro do peito do pé
<b>da</b>	Vristens omkreds
<b>fi</b>	Jalkapöydänympärys
<b>sv</b>	Runt vristen
<b>el</b>	Περίμετρος ράχης ποδιού (κουπέλι)
<b>cs</b>	Obvod nártu
<b>pl</b>	Obwód podbicia

<b>lv</b>	Pēdas pacēluma apkārtmērs
<b>it</b>	Kelties apimtis
<b>et</b>	Pahkluuümberrmõõt
<b>sl</b>	Obseg narta
<b>sk</b>	Obvod priehlavku
<b>hu</b>	Rúszet kerülete
<b>bg</b>	Обиколка на глезена
<b>ro</b>	Circumferința bolții piciorului
<b>ru</b>	Обхват подъема стопы
<b>hr</b>	Steznik za gležanj
<b>zh</b>	绕脚背一周

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## en

# REINFORCED PROPRIOCEPTIVE ELASTIC ANKLE BRACE

## Description/Destination

This device is a proprioceptive elastic ankle brace providing joint support. The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

## Composition

Polyamide - Elastodiene - Elastane - Polyester - Polyurethane - Silicone - Ethylen-vinyl acetate - Presence of natural latex.

## Properties/Mode of action

Elastic compressive knitting (class 3 - 20-36 mmHg / 271-48 hPa).

Silicone inserts and comfort zone on malleoli.

Supination brake via external reinforcement.

## Indications

Mild ankle sprain (acute phase).

Post-operative, post-traumatic residual pain and/or edema.

Joint instability/laxity.

Proprioceptive support for rehabilitation.

Sport activities resumption.

Compression for scar/burns healing.

## Contraindications

Do not use the product if the diagnosis has not been confirmed.

Do not apply the product in direct contact with broken skin or an open wound without an adequate dressing.

Do not use in the event of known allergy to any of the components.

History of venous or lymphatic disorders.

## Precautions

Verify the product's integrity before every use.

Do not use the device if it is damaged.

Choose the appropriate size to fit the patient, referring to the size chart.

It is recommended that a healthcare professional supervises the first application.

Strictly comply with your healthcare professional's prescription and recommendations for use.

In the event of discomfort, significant hindrance, pain, variation in limb volume, abnormal sensations or change in colour of the extremities, remove the device and consult a healthcare professional.

For hygiene and performance reasons, do not re-use the product for another patient.

Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

If the support is worn for prolonged periods of time, ensure that blood circulation is not impeded.

## Undesirable side-effect

This device can cause skin reactions (redness, itching, burns, blisters...) or wounds of various degrees of severity.

Possible risk of venous thrombosis.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

## Instructions for use/Application

Pull the ankle brace over the ankle joint, like a sock.

Place the malleoli correctly, on the comfort zones.

The silicone insert should not lie on the malleoli.

## Care/Maintenance

Product can be washed in accordance with the conditions shown on this notice and on the label. Hand wash. Do not use detergents, fabric softeners or aggressive products (products containing chlorine, etc.). Do not dry clean. Do not tumble-dry. Do not iron. Squeeze out excess water. Dry flat. Dry away from any direct heat source (radiator, sun, etc.).

## Storage

Store at room temperature, preferably in the original packaging.

## Disposal

Dispose of in accordance with local regulations.

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©Thuasne - 2023801 (2021-02)
Thuasne SAS – SIREN/RCS Nanterre 542 091 186 – capital 1 950 000 euros