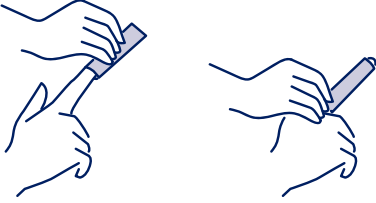
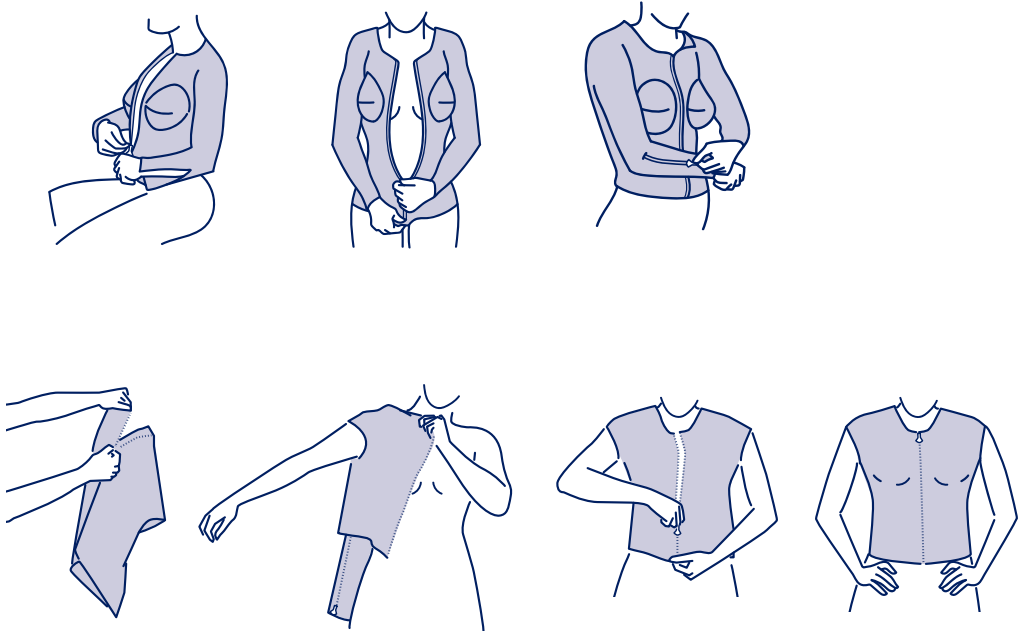
F

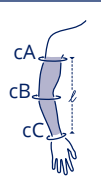
G

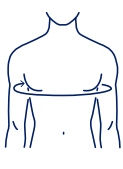
H

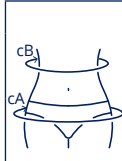
	cA	cB	cC	ℓ
	13 - 15 cm	20 - 25 cm	22 - 28 cm	39 cm
2	15 - 17 cm	23 - 28 cm	25 - 31 cm	
3	17 - 19 cm	26 - 31 cm	29 - 35 cm	
4	19 - 21 cm	29 - 34 cm	33 - 39 cm	
5	21 - 23 cm	32 - 37 cm	37 - 43 cm	
6	23 - 26 cm	35 - 41 cm	41 - 48 cm	

	cA	cB	cC	ℓ
en Lymphedema sleeve	Distal circumference	Medial circumference	Proximal circumference	Height
de Kompressionstrumpf für Lymphödem	Distaler Umfang	Medialer Umfang	Proximaler Umfang	Höhe
fr Manchon pour lymphoedème	Circonférence distale	Circonférence médiale	Circonférence proximale	Hauteur
nl Sleeve voor lymfoedeem	Distale omtrek	Mediale omtrek	Proximale omtrek	Hoogte
it Manicotto per linfedema	Circonferenza distale	Circonferenza mediale	Circonferenza prossimale	Altezza
es Manga para linfedema	Circunferencia distal	Circunferencia medial	Circunferencia proximal	Altura
pt Manga para linfedema	Circunferência distal	Circunferência medial	Circunferência proximal	Altura
da Muffe til lymfedem	Distal omkreds	Medial omkreds	Proksimal omkreds	Højde
fi Hiha lymfaturvotukseen	Distaalinen ympärysmitta	Mediaalinen ympärysmitta	Proksimaalinen ympärysmitta	Korkeus
sv Armstrumpa för lymfödem	Distal omkrets	Medial omkrets	Proximal omkrets	Höjd
el Μανίκι λεμφροδμήματος	Περιφερική περιμετρος	Μέση περιμετρος	Υψύς περιμετρος	Ύψος
cs Rukáv na lymfedém	Distální obvod	Mediální obvod	Proximální obvod	Výška
pl Rękaw na obrzęk limfatyczny	Obwód dystalny	Obwód przyśrodkowy	Obwód proksymalny	Wysokość
lv Uzrocis limfedēmai	Distālais apkārtmērs	Mediālais apkārtmērs	Proksimālais apkārtmērs	Augstums
lt Rankos mova limfedemai gydyti	Tolimojo galo apimtis	Vidurinės dalies apimtis	Artimojo galo apimtis	Aukštis
et Lymfödeemi varrukas	Distaalne ümbermõõt	Mediaalne ümbermõõt	Proksimaalne ümbermõõt	Kõrgus
sl Rokav za limfedem	Obseg distalno	Obseg medialno	Obseg proksimalno	Višina
sk Ramenný návlak na lymfedém	Distálny obvod	Obvod ramena cez lakeť	Obvod zápästia	Výška
hu Karharsisnya nyirokódémához	Disztális körméret	Mediális körméret	Proximális körméret	Hossz
bg Ръкав за лимфедем	Дистална обиколка	Медиална обиколка	Проксимална обиколка	Височина
ro Manșon pentru limfedem	Circumferință distală	Circumferință medială	Circumferință proximală	Înălțime
ru Рукав для лимфедемы	Дистальная окружность	Медиальная окружность	Проксимальная окружность	Высота
hr Rukav za limfoedem	Distalni opseg	Medijalni opseg	Proksimalni opseg	Visina
zh 淋巴水肿袖套	远端围	内侧围	近端围	高度
ja リンパ浮腫用スリーブ	遠位部の周径	中間部の周径	近位部の周径	高さ
ko 림프부종용 팔토시	팔 단 둘레	중간 부 둘레	근위 부 둘레	신장
ar كم للوذمات اللمفية	المحيط القاصي	المحيط المتوسط	المحيط الدائي	الارتفاع

	1	74 - 82 cm
	2	83 - 91 cm
	3	92 - 98 cm
	4	99 - 115 cm
	5	116 - 120 cm
	6	121 - 135 cm

en Sleeveless vest	Chest circumference
de Weste	Brustumfang
fr Gilet sans manche	Circonférence de la poitrine
nl Vest zonder mouwen	Borstomtrek
it Gilet senza maniche	Circonferenza del petto
es Chaleco	Contorno del pecho
pt Colete sem mangas	Circunferência do peito
da Ærmeless vest	Brystets omkreds
fi Hihaton liivi	Rinnan ympärysmitta
sv Väst utan ärmar	Omkrets thorax
el Αμόνικο γιλέκο	Περιμετρος στέρθους
cs Vesta bez rukávů	Obvod hrudníku
pl Kamizelka bez rękawów	Obwód klatki piersiowej
lv Veste bez piedurknēm	Krūškurvja apkārtmērs
lt Berankovė liemenė	Krūtinės apimtis
et Varrukateta vest	Rinna ümbermõõt
sl Telovnik brez rokavov	Obseg prsnega koša
sk Vesta bez rukávov	Obvod hrudníka
hu Ujjatlan mellény	Mellkas körméret
bg Жилетка без ръкави	Обиколка на гърдния кош
ro Vestă fără mâneci	Circumferința pieptului
ru Жилет без рукавов	Обхват грудной клетки
hr Prsluk bez rukava	Opseg grudi
zh 无袖背心	胸围
ja ノースリーブベスト	胸囲
ko 민소매 조끼	가슴둘레
ar صدر دون أكمام	محيط الصدر

	cA	cB
	1 88 - 94 cm	62 - 72 cm
	2 95 - 101 cm	67 - 79 cm
	3 102 - 108 cm	73 - 85 cm
	4 109 - 115 cm	80 - 93 cm
	5 116 - 122 cm	90 - 104 cm
	6 123 - 129 cm	97 - 113 cm

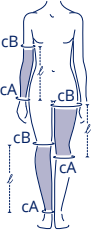
	cA	cB
en	Panty girdle / Mid-thigh short / Capri pants	Pelvis circumference
de	Miederhose/Bermudahose/Caprihose	Beckenumfang
fr	Culotte gainante / Panty / Corsaire	Circonférence du bassin
nl	Slip met korte pijpen / Pantybroek / Capribroek	Omtrek van het bekken
el	Guaina / Panty / Pantaloni capri	Circonferenza del bacino
es	Braga envolvente / Panty / Pirata	Contorno de la cadera
pt	Cueca modeladora / Panty / Corsário	Circunferência da bacia
da	Tætsluttende underbukser / Panty / knæbukser	Bækkenets omkreds
fi	Housuliivi / Alushousut / Kaprihousut	Lantion ympärysmitta
sv	Figurformande troska/cykelbyxa/trekvartsbyxa	Omkrets runt höften
el	Εσώρουχο συμπιέσας / Panty / Κολλάν capri	Περίμετρος λεκάνης
cs	Stahovací kalhotky / kalhoty / krátké kalhoty	Obvod pánve
pl	Majtki obcisujące / Panty / Spodnie typu capri	Obwód bioder
lv	Bikšītes / stilbikses / kapri bikses	ļegurņa apkārtmērs
it	Formuojamosios kelnaitės / kelnaitės / kelnės iki pusės blauzdu	Dubens apimtis
et	Vormivad aluspüksid / lühikesed püksid / retuusid	Puusa ümbermõõt
sl	Hlačke za oblikovanje / Hlačke / Tričetrtinske hlače	Obseg medenice
sk	Kompresívne nohavice / Legíny / Capri nohavice	Obvod panvy
hu	Hasleszorító nadrág / Nadrág / Capri nadrág	Csipőkörméret
bg	Оформящи боксерки / бельо / корсаж	Обиколка на таза
ro	Burtieră/Chilot/Pantaloni trei sferturi	Circumferința bazinului
ru	Компрессионные трусы / Шорты / Корсет	Обхват таза
hr	Gaće / kratke hlače / capri hlače	Opseg zdjelice
zh	束腰内裤 / 短裤袜 / 七分裤	骨盆围
ja	ガードルショーツ / パンティ / カプリパンツ	骨盤周り
ko	압박 팬티/팬티/압박바지	골반 둘레
ar	سروال تحتاني ضيق / باتني / بنطال كابرې	محيط الحوض

0	51 - 52 cm	en	Closed hood / open hood	Head circumference
1	53 - 55 cm	de	Geschlossene Kopfbandage/ Offene Kopfbandage	Kopfumfang
2	56 - 57 cm	fr	Cagoule fermée / cagoule ouverte	Circonférence de la tête
3	58 - 59 cm	nl	Gesloten bivakmuts / open bivakmuts	Omtrek van het hoofd
		it	Cappuccio / cappuccio aperto	Circonferenza della testa
		es	Capucha cerrada / capucha abierta	Contorno de la cabeza
		pt	Capuz fechado / capuz aberto	Circunferência da cabeça
		da	Lukket / åben elefanhue	Hovedets omkreds
		fi	Umpinainen huppu / aukinainen huppu	Pään ympärysmitta
		sv	Sluten huva/Öppen huva	Huvudets omkrets
		el	Κουκούλα πλήρους κάλυψης / ανοικτή κουκούλα	Περίμετρος κεφαλής
		cs	Zavřená kukla / otevřená kukla	Obvod hlavy
		pl	Kominiarka zamknięta / Kominiarka otwarta	Obwód głowy
		lv	Noslēgta kapuce / atvērta kapuce	Galvas apkārtmērs
		it	Uždaras gobtuvas / atviras gobtuvas	Galvos apimtis
		et	Suletud kapuuts / avatud kapuuts	Pea ümbermõõt
		sl	Balaklava zaprta / balaklava odprta	Obseg glave
		sk	Bandáž hlavy zatvorená / Bandáž hlavy otvorená	Obvod hlavy
		hu	Zárt maszk / nyitott maszk	Fej körméret
		bg	Затворена/отворена гугла	Обиколка на главата
		ro	Cagulă închisă/cagulă deschisă	Circumferința capului
		ru	Закрытая маска/открытая маска	Окружность головы
		hr	Zatvorena kapuljača / otvorena kapuljača	Opseg glave
		zh	闭合式头罩 / 开放式头罩	头围
		ja	フルフェイスフード / オープンフェイスフード	頭囲
		ko	전면형 복면/ 오픈형 복면	머리둘레
		ar	قناع رأس مغلق / قناع رأس مفتوح	محيط الرأس

	1	27 - 30 cm
	2	31 - 36 cm
	3	37 - 45 cm

en	Partial hood	Neck circumference
de	Kinnschlinge	Halsumfang
fr	Cagoule partielle	Circonférence du cou
nl	Gedeeltelijke bivakmuts	Omtrek van de hals
it	Sappuccio parziale	Circonferenza del collo
es	Capucha parcial	Contorno del cuello
pt	Capuz parcial	Circunferência do pescoço
da	Halv elefanthue	Halsens omkreds
fi	Osittainen huppu	Kaulan ympärysmitta
sv	Partiell huv	Halsens omkrets
el	Κοκούλα μερικής κάλυψης	Περίμετρος λαιμού
cs	Částečná kukla	Obvod krku
pl	Kominiarka częściowa	Obwód szyi
lv	Dalēja kapuce	Kakla apkārtmērs
lt	Dalinis gobtuvas	Kaklo apimtis
et	Poolkapuuts	Kaela ümbermõõt
sl	Balaklava delna	Obseg vratu
sk	Čiastočná bandáž hlavy	Obvod krku
hu	Részleges maszk	Nyak körmért
bg	Частична гугла	Обиколка на шията
ro	Sagulă parțială	Circumferința gâtului
ru	Частичная маска	Обхват шеи
hr	Polu-karuljača	Opseg vrata
zh	局部头罩	脖圍
ja	パーシャルフード	首周り
ko	부분 목면	목 둘레
ar	قناع رأس جزئي	محيط العنق

		cA	cB	l
	1	9 - 11 cm	19 - 23 cm	35 cm
	2	11 - 14 cm	23 - 28 cm	
	3	14 - 16 cm	28 - 32 cm	
	4	16 - 20 cm	32 - 37 cm	
	5	20 - 25 cm	37 - 42 cm	45 cm
	6	25 - 30 cm	42 - 48 cm	
	7	30 - 40 cm	48 - 58 cm	
8	40 - 50 cm	58 - 67 cm		

		cA	cB	l
en	Long sleeve	Distal circumference	Proximal circumference	Height
de	Langer Kompressionstrumpf	Distaler Umfang	Proximaler Umfang	Höhe
fr	Manchon long	Circonférence distale	Circonférence proximale	Hauteur
nl	Lange sleeve	Distale omtrek	Proximale omtrek	Hoogte
it	Manicotto lungo	Circonferenza distale	Circonferenza prossimale	Altezza
es	Manga larga	Circunferencia distal	Circunferencia proximal	Altura
pt	Manga longa	Circunferência distal	Circunferência proximal	Altura
da	Lang mufte	Distal omkreds	Proksimal omkreds	Højde
fi	Pitkä lahje/hiha	Distaalinen ympärysmitta	Proksimaalinen ympärysmitta	Korkeus
sv	Lång ärm	Distal omkrets	Proximal omkrets	Höjd
el	Μακρύ μανίκι	Περιφερική περίμετρος	Εγγύς περίμετρος	Ύψος
cs	Dlouhý rukáv	Distální obvod	Proximální obvod	Výška
pl	Długi rękaw	Obwód dystalny	Obwód proksymalny	Wysokość
lv	Garais uzrocis	Distālais apkārtmērs	Proksimālais apkārtmērs	Augstums
lt	Ilga mova	Tolimojo galo apimtis	Artimojo galo apimtis	Aukštis
et	Pikk varrukas	Distaalne ümbermõõt	Proksimaalne ümbermõõt	Kõrgus
sl	Rokav dolg	Obseg distalno	Obseg proksimalno	Višina
sk	Ramenný návlak dlhý	Distálny obvod	Obvod zápästia	Výška
hu	Hosszú szár	Disztális körmért	Proximális körmért	Hossz
bg	Дълъг ръкав	Дистална обиколка	Проксимална обиколка	Височина
ro	Manșon lung	Circumferință distală	Circumferință proximală	Înălțime
ru	Длинный рукав	Дистальная окружность	Проксимальная окружность	Высота
hr	Dugi rukav	Distalni opseg	Proksimalni opseg	Višina
zh	长袖套	远端圍	近端圍	高度
ja	ロングスリーブ	遠位部の周径	近位部の周径	高さ
ko	긴 소매	말단 둘레	근위부 둘레	신장
ar	كم طويل	المحيط القاصي	المحيط الداني	الارتفاع

	1	40 - 52 cm
	2	52 - 64 cm

en	Ear band	Head circumference
de	Kopfband	Kopfumfang
fr	Bandeau pour oreilles	Circonférence de la tête
nl	Oorband	Omtrek van het hoofd
it	Fascia per le orecchie	Circonferenza della testa
es	Banda para las orejas	Contorno de la cabeza
pt	Faixa para as orelhas	Circunferência da cabeça
da	Ørebånd	Hovedets omkreds
fi	Korvaside	Pään ympärysmitta
sv	Öronband	Huvudets omkrets
el	Κορδέλα αυτιών	Περίμετρος κεφαλής
cs	Bandáž na uši	Obvod hlavy
pl	Opaska na uszy	Obwód głowy
lv	Ausu saite	Galvas apkārtmērs
lt	Ausų tvarstis	Galvos apimtis
et	Kõrvapael	Pea ümbermõõt
sl	Trak za ušesa	Obseg glave
sk	Bandáž uší	Obvod hlavy
hu	Fülpánt	Fej körméret
bg	Ушанки	Обиколка на главата
ro	Bentîţă pentru urechi	Circumferinţa capului
ru	Повязка на уши	Окружность головы
hr	Zavoj za uši	Opseg glave
zh	护耳束头带	头围
ja	耳用バンド	頭囲
ko	이어머프 헤드밴드	머리둘레
ar		محيط الرأس

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en
STANDARD AND PATIENT-SPECIFIC COMPRESSION DEVICE

Description/Destination

The Cicatrex range of compression devices is available in standard and patient-specific versions, as described in the tables below.

- Cicatrex Standard compression devices are available in several designs and fabrics:

Parts of the body	Models	Fabrics
Leg	<ul style="list-style-type: none"> • Panty girdle • Mid-thigh short • Capri pants • Trouser 	Nature
Arm	<ul style="list-style-type: none"> • Lymphedema sleeve 	Skin
Arm or leg	<ul style="list-style-type: none"> • Long sleeve • Short sleeve 	Nature
Foot	<ul style="list-style-type: none"> • Open foot glove 	Skin
Hand	<ul style="list-style-type: none"> • Microfiber glove 	Fine
	<ul style="list-style-type: none"> • Finger protector • Long-sleevelet closed-finger ladder-proof glove • Short-sleevelet closed-finger ladder-proof glove • Short-sleevelet open-finger ladder-proof glove 	Skin
Head	<ul style="list-style-type: none"> • Closed hood • Partial hood • Open hood • Ear band 	Nature
Thorax	<ul style="list-style-type: none"> • Sleeveless vest 	Nature

Refer to size charts.

- Cicatrex patient-specific compression devices are available in several designs and fabrics:

Parts of the body	Models	Fabrics
Leg	<ul style="list-style-type: none"> • Mid-thigh short • Half-tights • Tights • Thigh-high • Sock 	Nature Fine Skin
Arm	<ul style="list-style-type: none"> • Armsleeve 	Nature Fine Skin
Foot	<ul style="list-style-type: none"> • Foot glove 	Fine Skin
Hand	<ul style="list-style-type: none"> • Finger protector • Mitten • Glove 	Fine Skin
Head	<ul style="list-style-type: none"> • Hood • Chinstrap 	Fine Skin
Thorax	<ul style="list-style-type: none"> • Shoulder brace • Vest • Jacket • Bodysuit 	Nature Fine Skin

Different finishing options are available according to your healthcare professional's recommendations.

- The Cicatrex models for Ehlers-Danlos syndrome are available: short sleeve, long sleeve, short leg, long leg, high open sock, high closed sock, forearm sleeve, full arm sleeve, sleevelet, hand glove, mitten, Mid-thigh short, sleeveless vest, leg sleeve, foot glove.

Use: the device is intended for use on the lower limbs, upper limbs, thorax or head of adult or pediatric patients suffering from at least one of the indications listed below.

Composition

Cicatrex Fine and Nature fabrics: viscose - polyester - elastane.

Cicatrex Skin fabric: polyamide - elastane - polyester - cotton.

Finishes: polyamide - elastane - polyester - cotton - viscose.

Properties/Mode of action

The Cicatrex compression principle helps reorganize collagen fibers in a plane parallel to the skin, thus limiting uncontrolled scarring of connective tissue.

Unsightly scars and dysfunctional adhesions are reduced by compression.

Cicatrex compression devices provide a minimum compression of 10 mmHg.

Models for Ehlers-Danlos syndrome provide compression of between 6 and 13.5 mmHg.

Indications

Treatment of burns (thermal, chemical or electrical).

Prevention of functional or unsightly scar complications (keloids, hypertrophy, adhesions).

Support for skin grafts.

Treatment of acute/postoperative edema (e.g.: breast surgery) or chronic edema (e.g.: lymphedema).

Treatment of symptoms and/or disorders associated with Ehlers-Danlos syndrome: joint pain, dislocation, incomplete joint dislocation, motor instability (only for models linked to this indication).

Prevention of chemotherapy-induced peripheral neuropathy of the extremities (for hand gloves and socks only).

Contraindications

Do not use in case of major dermatological conditions on an affected limb.

Do not use in the event of known allergy to any of the components.

Do not use in case of severe peripheral neuropathy of the affected limb.

Do not use in the event of septic thrombosis.

Do not use in the event of phlegmasia cerulea dolens (painful blue phlebitis with arterial compression) of an affected limb.

Specific to the lower limbs:

Do not use in case of lower limb peripheral arterial disease (PAD) with ankle-brachial index (ABI) < 0.6.

Do not use in the event of decompensated heart failure.

Do not use in case of extra-anatomical bypass of the affected limb.

Specific to the upper limbs:

Do not use in case of brachial plexus condition.

Do not use in case of vasculitis of the extremities.

Specific to head or neck models:

Do not use in cases of carotid stenosis or stenosis of the vessels of the neck and head.

Specific to models applied to the abdomen:

Do not use for pregnant women (from the first month).

Precautions

Strictly comply with your healthcare professional's prescription and recommendations for use.

The risk/benefit ratio and the adequate level of pressure must be regularly re-assessed by a healthcare professional in the event of:

- PAD with ABPI between 0.6 and 0.9,
- Advanced peripheral neuropathy,
- Weeping or eczematous dermatitis.

Take care not to damage the device when fitting it, particularly with your nails. For hygiene and performance reasons, do not re-use the product for another patient.

In the event of discomfort, significant restriction, variation in limb volume or abnormal sensations, deterioration in skin condition, infection, change in extremity color, or change in device performance, remove the device and seek the advice of a healthcare professional.

Do not apply the product directly to injured skin or open wounds without a suitable dressing.

Check that the anti-slip system can be applied to healthy, unbroken skin.

In the case of closed garments, check that the hook-and-loop fastener is not in direct contact with the skin, as it may cause irritation.

To avoid injury during use, make sure that the seams are always visible on the outside of the garment.

Do not use when swimming.

For the pantyhose model, if worn with compression stockings, we recommend placing the device over the stockings.

In the case of diffuse or large-area burns, use a patient-specific garment.

To prevent peripheral neuropathy, the garment should be worn before the chemotherapy injection.

In the case of chronic or progressive disease, or changes in morphology, it may be necessary to adapt or renew the compression device in order to maintain the expected level of compression. Regular follow-up is recommended.

Seek the advice of a healthcare professional in the following cases:

- Before using a hood or neck compression garment.
- For dry skin.
- In case of pelvic floor deficiency.
- In case of hiatal or inguinal hernia.

In the case of garments worn on the thorax/abdomen, the restrictive effect of the device could alter respiratory function and reduce lung capacity and tidal volume.

In children, permanent compression can lead to growth problems (mainly in the head area). Monitoring should be more regular, and clothing may need to be adapted to suit growth.

In the event of damage to or tearing of the compression garment, it should be replaced.

Adverse side-effects

This device can cause skin reactions (redness, itching, burns, blisters...) or wounds of various degrees of severity or cutaneous dryness.

Models applied to the abdomen can lead to gastroesophageal reflux or hernia aggravation.

Any serious device-related incidents occurring should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use/Application

Choose the appropriate size to fit the patient, referring to the size chart; not applicable to the medical devices made to measure patient, the measures are individually taken.

Verify the product's integrity before every use.
Cicatrex compression devices should be worn directly on the skin.
Ask a health professional for advice.
Avoid pulling on the garment more than is necessary.
Adjust the garment to ensure it is not creased.
Once the garment is completely in place, massage the covered body area to ensure even compression.

Ⓐ Putting on the sleeves:

- Open the zipper or hook-and-loop fastener if the garment has one.
- Insert the hand into the garment and turn the garment back on itself up to the opening provided for the hand.
- Insert the hand into the garment then slide the garment carefully along the limb up to the armpit.
- Close the zipper or hook-and-loop fastener if the garment has one.

Ⓑ Putting on gloves and mittens:

- Open the zipper or hook-and-loop fastener if the garment has one.
- Place the hand inside the garment, then carefully slide it up to the wrist (short sleevelet) or elbow (long sleevelet).
- Close the zipper or hook-and-loop fastener if the garment has one.

Ⓒ Putting on foot gloves:

- Insert the foot into the garment then slide it carefully up to the ankle.
- Scissors can be used to cut the garment to the desired length.

Ⓓ Putting on socks:

- Open the zipper or hook-and-loop fastener if the garment has one.
- Insert the foot into the garment until the heel is correctly positioned.
- Carefully slide the garment up the limb to below the knee (sock).
- Close the zipper or hook-and-loop fastener if the garment has one.

Ⓔ Putting on thigh-high stockings, mid-thigh shorts, capri pants, panty girdles, tights and half-tights:

- Open the zipper or hook-and-loop fastener if the garment has one.
- Insert the hand into the garment and turn the garment back on itself up to the heel.
- Insert the foot into the garment until the heel is correctly positioned.
- Carefully slide the garment up the limb to below the buttock crease (thigh-high stockings) or to the waist (mid-thigh short, capri pants, panty girdle, half-tights, tights).
- Close the zipper or hook-and-loop fastener if the garment has one.

Ⓕ Putting on hoods, ear bands and chinstraps:

- Place the head inside the garment, then carefully slide it to the neck (hood and chinstrap) or to the ears (ear band).
- The closed and open hoods should be cut with scissors around the appropriate rectangles to allow the ears to pass through.

Ⓖ Putting on finger protectors:

- Place the finger inside the garment, then carefully slide it to the base of the finger.

Scissors can be used to cut the garment to the desired length.

Ⓖ Putting on vests, jackets and shoulder braces:

- Open the zipper or hook-and-loop fastener if the garment has one.
- Insert the arm(s) into the garment, then carefully slide it up the arm(s) to the shoulders.
- Close the zipper or hook-and-loop fastener if the garment has one.

Ⓗ Putting on bodysuits:

- Open the zipper or hook-and-loop fastener if the garment has one.
- Insert the legs into the garment, then carefully slide it up to the top of the torso.
- Insert the arms into the garment's sleeves.
- Close the zipper or hook-and-loop fastener if the garment has one.

Care

We recommend washing your new garment before use. We recommend washing your garment every day, or after each use. Cicatrex compression devices are designed for daily use over a 6-month period (i.e. 90 washes). After this time, garments should be replaced to ensure the treatment is effective and technical performance is not affected.

Washing:

Machine wash at 30°C (delicate programme). Use a laundry detergent without fabric softener. Do not use aggressive products (e.g.: chlorinated products). Put the garment in a washing net to prevent other garments (especially those with hook-and-loop fasteners or zippers) from damaging your brace. Wring out without twisting. Dry away from any direct heat source (radiator, sun, etc.). Do not tumble-dry. Do not iron.

Storage

Store at room temperature, out of direct sunlight.

Disposal

Dispose of in accordance with local regulations.

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© Trä på fothandske:
THUASNE UK Ltd**
Thuasne SAS - SIREN/RCS Nanterre 542 091 186