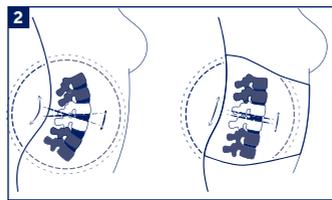
THUASNE

Lombax Original

- (fr)** Ceinture de soutien lombaire
- (en)** Lumbar support belt
- (de)** LWS-Stabilisierungsothese
- (nl)** Lumbale rugbrace
- (it)** Cintura di sostegno lombare
- (es)** Faja de sujeción lumbar
- (pt)** Cinta de suporte lombar
- (da)** Rygstøttebælte
- (fi)** Lannetukivyö
- (sv)** Ryggstöd för ryggen
- (no)** Støttebelte for korsryggen
- (el)** Ζώνη οσφυϊκής υποστήριξης
- (cs)** Bederní podpurný pás
- (pl)** Podtrzymujący pas lędźwiowy
- (lv)** Muguras jostas daļas atbalsta josta
- (lt)** Palaikomasis juosmens diržas
- (et)** Nimmepiirkonna tugikorsett
- (sl)** Oporni križni pas
- (sk)** Podporný driekový pás
- (hu)** Ágyéki fűző
- (bg)** Колан за поддържане на кръста
- (ro)** Centură de susținere lombară
- (ru)** Опорный поясничный корсет
- (hr)** Lumbalni potporni pojas
- (zh)** 加强型腰部稳定护带
- (ja)** 腰サポートベルト
- (ar)** حزام لأسفل الظهر

| 3 | 4 * |
|---|--------------|
| | 60 - 85 cm |
| | 86 - 112 cm |
| | 113 - 140 cm |

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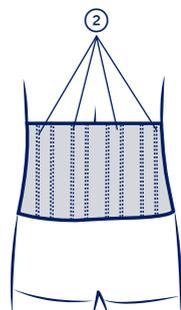
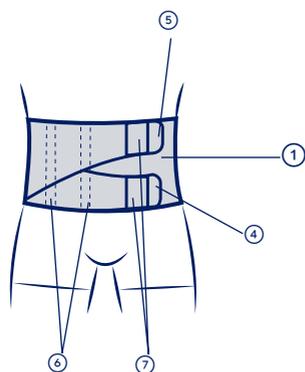
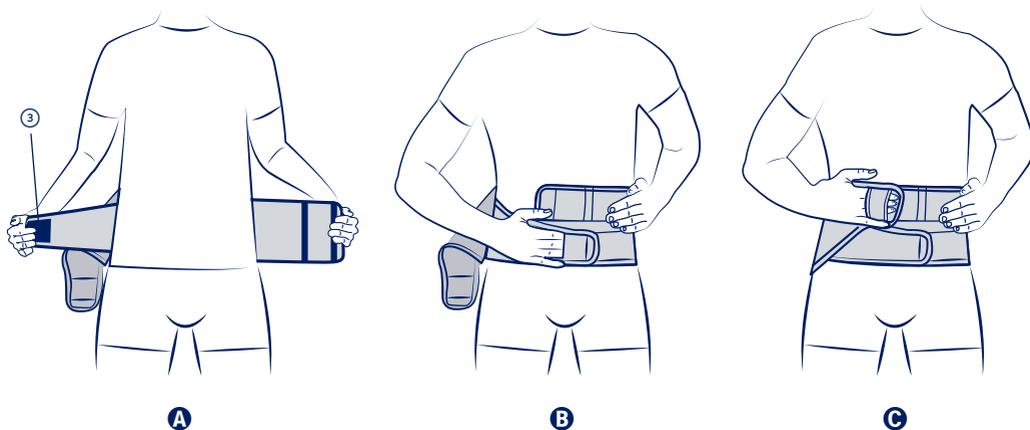
LUMBAR SUPPORT BELT

Description

- Elastic fabric ① (polyamide, polyester, elastane, viscose).
- Four reshapable stays ② (brushed tempered steel).
- Two abdominal panels with touch fastening system ③ (polyamide) with independent lower ④ and upper ⑤ straps.
- Two flexible stays ⑥ (polyoxymethylene).
- Two finger loops ⑦ (polyester).
- 3D knit back panel (polyester).
- Edging (polyester, elastodiene, latex).

Mechanism of action

The **Lombax Original** belt relieves low back pain, supporting the lumbar spine: compartmental lumbar unloading is therefore achieved via increased intra-abdominal pressure.

Indications

Adult patients with the conditions indicated and compatible with the size chart*:

- Non-specific low back pain (acute, subacute and chronic phases)
- Lumbar stenosis

Destination:

This brace should only be used for back treatment.

Fitting instructions

- Choose the appropriate size to fit the patient, referring to the size chart (indicated on the box and on the instruction leaflet).
- Before use, ensure that the product is not damaged.
- Wear the belt on top of a t-shirt or thin vest.
- Open the belt and lay it out completely flat so that you can see all of the components.
- Position the belt around your waist, placing your right hand through the finger loop ⑦ located on the right lower strap ④Ⓐ. The belt must be centred over the spine. The bottom of the belt must stop at the top of the crease between the buttocks. The Thuasne logo indicates which way up the belt should be put on. The Thuasne name must be the right way up.
- Fasten the belt using the wide touch fastening straps: attach the lower strap ④Ⓑ on the left side, adjusting to the desired tightness. Repeat the procedure with the upper strap ⑤Ⓒ.
- If necessary, adjust the tightness to suit your requirements.

Contraindications

- Do not use on pregnant women.
- Do not use in the event of known allergy to any of the materials (see product composition in the Description section).
- Do not use in the event of a hiatus hernia.
- Do not use in the event of bone cancer with spinal metastases.
- Do not use in the event of deficient respiratory functions.

Precautions

It is recommended that a health professional adjust the stays to the shape of the patient's back before the belt is worn for the first time.

For temporary use, combined with functional rehabilitation if necessary, following the advice or prescription of a health professional.

Consult a health professional if discomfort or pain occurs. Do not use the device if it is damaged.

Do not wear the product while sleeping.

Do not wear the product in a medical imaging machine.

Store the device at room temperature, in its original packaging.

Reusable medical device. For hygiene and performance reasons, do not re-use the product for another patient.

Dispose of the product with household waste (no special disposal conditions).

Unwanted side effects

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Washing instructions

- Fasten the touch fastening system before washing.
- Machine wash at a maximum temperature of 40°C (forty degrees Celsius), using the delicates programme.
- Do not machine wash the device more than 10 times. Do not dry clean.
- Do not wash the device using aggressive products containing chlorine or oxygenated whitening products.
- Do not tumble dry. Do not iron.
- Squeeze out excess water and dry flat away from heat sources.

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Thuasne Benelux B.V.
Thuasne Scandinavia AB
Thuasne Cervitex Ltd
Thuasne UK Ltd
Thuasne (Shanghai) Medical Devices Co., Ltd.