# THUASNE

# SPORT

<b>fr</b>	<b>Poignet élastique Thuasne Sport</b> .....	<b>3</b>
<b>en</b>	<b>Thuasne Sport elastic wrist support</b> .....	<b>4</b>
<b>de</b>	<b>Elastische Handgelenkbandage Thuasne Sport</b> .....	<b>5</b>
<b>nl</b>	<b>Thuasne Sport elastische polsorthese</b> .....	<b>7</b>
<b>it</b>	<b>Polsiera elastica Thuasne Sport</b> .....	<b>8</b>
<b>es</b>	<b>Muñequera elástica Thuasne Sport</b> .....	<b>9</b>
<b>pt</b>	<b>Punho elástico Thuasne Sport</b> .....	<b>11</b>
<b>da</b>	<b>Elastisk armbånd Thuasne Sport</b> .....	<b>12</b>
<b>fi</b>	<b>Joustava Thuasne Sport -rannetuki</b> .....	<b>13</b>
<b>sv</b>	<b>Thuasne Sport elastisk handled</b> .....	<b>14</b>
<b>el</b>	<b>Ελαστικό περικάρπιο Thuasne Sport</b> .....	<b>15</b>
<b>cs</b>	<b>Elastická bandáž zápěstí Thuasne Sport</b> .....	<b>17</b>
<b>pl</b>	<b>Elastyczna orteza na nadgarstek Thuasne Sport</b> .....	<b>18</b>
<b>lv</b>	<b>Thuasne Sport elastīgā aproce</b> .....	<b>19</b>
<b>lt</b>	<b>Tamprus riešo įtvaras „Thuasne Sport“</b> .....	<b>21</b>
<b>et</b>	<b>Thuasne Sport elastne randmetugi</b> .....	<b>22</b>
<b>sl</b>	<b>Elastičen zapestni trak Thuasne Sport</b> .....	<b>23</b>
<b>sk</b>	<b>Elastický náramok Thuasne Sport</b> .....	<b>24</b>
<b>hu</b>	<b>Thuasne Sport rugalmas csuklórögzítő</b> .....	<b>26</b>
<b>bg</b>	<b>Еластичен накитник Thuasne Sport</b> .....	<b>27</b>
<b>ro</b>	<b>Orteză elastică pentru încheietura mâinii Thuasne Sport</b> .....	<b>28</b>
<b>ru</b>	<b>Эластичный ортез на запястье Thuasne Sport</b> .....	<b>30</b>
<b>hr</b>	<b>Elastični steznik za zglob Thuasne Sport</b> .....	<b>31</b>
<b>zh</b>	<b>Thuasne Sport弹性腕部护具</b> .....	<b>32</b>
<b>ar</b>	<b>Thuasne Sport تجهيزة الرسغ المرنة</b> .....	<b>35</b>

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---

## en

# THUASNE SPORT ELASTIC WRIST SUPPORT

## Description/purpose

This device is an elastic wrist orthosis providing joint support. The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

## Composition

Polyamide - Elastane - Synthetic rubber (CR).

## Properties/Mode of action

Adjustable fastening system.  
Bilateral product.

## Indications

Wrist sprain.  
Wrist tendinopathies.

## Contraindications

Do not apply the product in direct contact with broken skin.  
Do not use in the event of known allergy to any of the components.

## Precautions

Verify the product's integrity before every use.  
Do not use the product if it is damaged.  
Choose the appropriate size to fit the patient, referring to the size chart. It is recommended that a healthcare professional supervises the first application.  
Strictly comply with your healthcare professional's prescription and recommendations for use.

Consult a healthcare professional:

- In case of doubt before use,
- In case of uncertain diagnosis

Consult a healthcare professional and stop using the device:

- In case of discomfort, significant hindrance, pain, abnormal sensations,
- In case of changes in the device's performance.

For hygiene, security and performance reasons, do not re-use the product for another patient.

Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

It is recommended to adequately tighten the device to achieve support/immobilisation without restricting blood circulation.

## Undesirable side-effects

This device can cause skin reactions (redness, itching, burns, blisters, etc.) or wounds of various degrees of severity. Any serious incidents related to the product should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

## Instructions for use/Application

- Open the hook-and-loop fasteners.
- Place the support on the back of the hand and insert the thumb.
- Tighten the support using the hook-and-loop fasteners.

## Care

Product can be washed in accordance with the instructions shown on this leaflet and on the label. Hand wash. Close the self-fastening tabs before washing. Do not use detergents, fabric softeners or aggressive products (products containing chlorine). Do not dry clean. Do not tumble-dry. Do not iron. Squeeze out excess water. Dry flat. Dry away from any direct heat source (radiator, sun, etc.).

## Storage

Store at room temperature, preferably in the original packaging.

## Disposal

Dispose of in accordance with local regulations.  
Keep this instruction leaflet

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©Thuasne - 2034001 (2022-02)
Thuasne SAS – SIREN/RCS Nanterre 542 091 186