



THUASNE

de Gebrauchsanweisung
en Instructions for use
fr Notice d'utilisation
it Istruzioni per l'uso
es Instrucciones de utilización
pt Instruções de uso
nl Gebruiksaanwijzing
da Brugsanvisning
sv Bruksanvisning
fi Käyttöohje
no Bruksanvisning
el Οδηγίες χρήσης
cs Návod k použití
pl Sposób użycia
et Kasutusjuhend
lv Lietošanas instrukcija
lt Naudojimo instrukcija
sk Návod na použitie
sl Navodila za uporabo
hu Használati utasítás
ro Instrucţiuni de utilizare
bg Инструкция за употреба
hr Uputa za uporabu
mk Упатство за користење
sr Uputstvo za korišćenje
tr Kullanım Talimatları
ru Инструкция по применению
ko 사용 설명서
ja 使用方法
he הוראות שימוש
ar دليل الاستخدام حزام
zh 使用说明

SILIMA®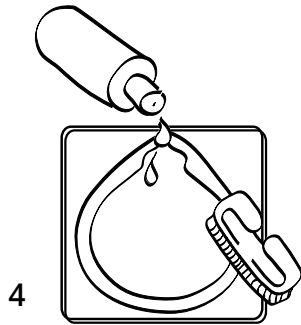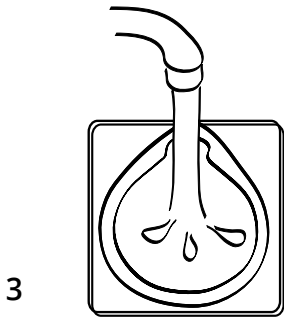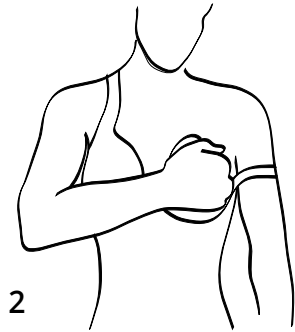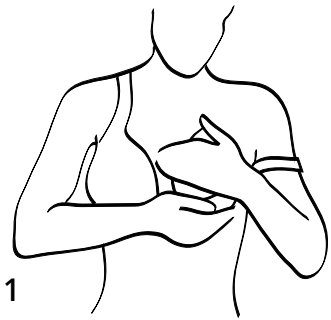

de	4/5	lt	27/28
en	5/6	sk	29/30
fr	7/8	sl	30/31
it	8/9	hu	32/33
es	10/11	ro	33/34
pt	11/12	bg	35/36
nl	13/14	hr	36/37
da	14/15	mk	38
S	16/17	sr	39
fi	17/18	tr	40
no	19	ru	41
el	20/21	ko	42
cs	21/22	ja	43
pl	23/24	he	44
et	24/25	ar	45
lv	26/27	zh	46

SILIMA®-Brustepithesen (T66107, T66110, T66328, T66360, T66368, T66370, T66374, T66375, T66377, T66378, T66379, T66380, T66387, T66390, T66398)

Anwendung

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How to look after SILIMA®

You have selected a quality product in choosing a SILIMA® breast form. The soft, skin-friendly material has excellent properties when worn: consistency, elasticity and movement are very similar to the natural properties of the breast.

Instructions for Use

All SILIMA® breast forms should be worn in a suitable bra, either pocketed or directly against the skin.

Indication and intended purpose

After breast surgery (mastectomy, breast-conserving surgery, breast reconstruction) and after full healing of the surgical wound and completed radiation therapy, SILIMA® breast prostheses and SILIMA® compensation prostheses will give you a natural, cosmetically and visually harmonious appearance as well as a corresponding weight compensation. Design and size depend on anatomical conditions. Consistency, elasticity and behaviour in motion are very similar to the characteristics of the natural breast. All materials used have been tested for skin tolerance.

Contraindications

None known

Instructions for Care

Daily care: Clean the SILIMA® breast form in luke warm water using a mild detergent or soap. Never use a brush! To dry, wrap the SILIMA® breast form in a hand towel and pat it lightly till dry. Do not place the form on a heater and never use a hair-dryer!

Important Points

Keep pointed and sharp objects away from the breast form. These can easily damage the delicate outer skin and make the form unserviceable. A repair is not possible. Avoid dropping the form onto a hard surface. Please always store your SILIMA® in its original packaging!

SILIMA® breast forms are resistant to chlorine and salt water. In rare cases, the consistency of the silicone may be affected by aggressive perspiration and bacteria. A slight loss in shape and discolouration may occur with time.

en

Product Warranty

This SILIMA® silicone product carries a 2-year warranty against manufacturing faults. Please make a note of the date of receipt to validate any future claim.

In the unlikely event of any fault occurring please return the breast form in its original packaging to your hospital or other supplier who will process the claim on your behalf.

Please note that this warranty does not cover products that have been incorrectly stored or exposed to damaging external influences, e.g., sharp objects, pet's claws, etc., or extend to adhesive pads and strips. We comply with all statutory requirements and regulations.

Precautions

Follow the advice of the professional who prescribed or supplied the product. Consult your professional if discomfort occurs. Store at room temperature, preferably in its original packaging. For hygiene and performance reasons, do not re-use the product for another patient. Any serious incident that occurs in connection with the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient are/is domiciled.

Instructions for Use of self-adhesive products

SILIMA® direct, SILIMA® shell direct and SILIMA® Xtra (T66377, T66387 and T66107)

How to apply the adhesive pad/strips to the breast form:

- The area around the breast and the pad or strips must be dry and free of any grease
- Tear open the foil pouch at the indentation and remove the pad or strips from the wrapper
- N.B.: For SILIMA® direct pads T66377 only – Remove the white cover and keep in a safe place. Important: Do not remove the label with the picture symbol!
- Fit the pad/strips inside the raised edges, and gently squeeze for a few seconds to ensure good adhesion

How to apply the adhesive breast form to your body

- Hold the rounded front of the form in one hand, and remove the release liner(s) with the other. Please retain and store liner(s) for future use
- Press the SILIMA® form against your body for a few seconds **(1)**
- Put on and position bra
- N.B.: In the case of SILIMA® direct T66377, it might be helpful to put on the bra beforehand, slip off the shoulder strap to provide access, place the form and then reposition the bra strap

How to remove the adhesive breast form

- Gently peel off the breast form from either side, never from the top **(2)**
- Store product in its original box, covering the pads/strips with the release liner provided

Cleansing of the adhesive pad/strip

- Place the pad/strips on the cleaning board and remove the cover(s)
- Dampen thoroughly with luke-warm water **(3)**
- Add one to two drops of the SILIMA® cleansing lotion to the adhesive surface. Please only use this cleansing lotion as adhesion can otherwise not be guaranteed! **(4)**
- Gently clean the pad/strips with the wet SILIMA® brush, provided for this purpose **(4)**
- Lightly brush and rinse under running water; continue until the surface is 'roughened'
- Clean the other side as well if required, by turning pad/strips over on the cleaning board

To dry, place the cleaning board at an angle with the draining ridge at the bottom, or use a hair dryer on a low setting from a safe distance of approx. 30 cm/12".

Technical data for all silicone prostheses

Polyurethane film, material on the basis of polydimethylsiloxane, colour pigments

Additionally with T66 328, 378, 368, 398, 377, 374, 375 (lightweight prostheses): Low weight plastic hollow microspheres

Additionally with T66 377, 387, 107 (adhesive products): Self-adhesive material on the basis of polydimethylsiloxane

Cleansing Lotion

for SILIMA® direct Adhesion Pad, 1-2 drops are sufficient
 Contains: 5-15% anionic tensides, 1-5% amphoteric tensides, cleansing agents, aromatic components, colouring, preservatives

Prothèses mammaires SILIMA® (T66107, T66110, T66328, T66360, T66368, T66370, T66374, T66375, T66377, T66378, T66379, T66380, T66387, T66390, T66398)

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- S** THUASNE SCANDINAVIA AB · Box 95 · 131 07 Nacka
- CHN** THUASNE (Shanghai) Medical Devices Co., Ltd. · 159 Tianzhou Lu · Shanghai, 200233
Thuasne UK Ltd
Thuasne Deutschland GmbH